



Bullen.



Prescription Codes



Contains: 1 x briefs, 1 x flange plate, 10 non-return valve pouches and 1 night/leg bag connector

20mm	ABPlus/18/20		
25mm	ABPlus/18/25	32.5mm	ABPlus/18/32.5
27.5mm	ABPlus/18/27.5	35mm	ABPlus/18/35
30mm	ABPlus/18/30	40mm	ABPlus/18/40

When requesting the Acti brief Plus please provide the exact waist size in order for us to provide the correct size briefs.

Brief Size -	Small	28-33 inch hip	XLarge	40-44 inch hip
	Medium	34-36 inch hip	XXLarge	45-50 inch hip
	Large	37-40 inch hip		

*When washing the Acti brief Plus, please keep the flange plate in place, and wash at no higher than 30 degrees or alternatively hand wash.

Spare pouches

SPlus18 contains 15 x non-return valve pouches
SP18 contains 15 x pouches

For more information and to also find out about Bullen Healthcare Home Delivery Service please contact us on Freephone **0800 888 501**



Bullen.

Bullen Healthcare Ltd.
Glacier Buildings,
Brunswick Business Park,
Harrington Road, Liverpool L3 4BH
Freephone: 0800 888 501
info@bullens.com www.bullens.com



bullens.com
follow us on
facebook
and twitter



RE-ORDER REF: 306033

Delivering Innovation in Product Development

The Acti-Brief Plus is a unique and innovative Male Urinary Continence Management System and has been designed with the needs of your patient in mind.

Comfort

Soft cotton pants with a unique and modern design

Discreet

Providing reassurance for a condition that can often cause embarrassment for patients

Independence

As healthcare providers we have a duty to focus on the quality of life of each individual

Freedom

We design products that work and in doing so provide freedom and peace of mind

www.bullens.com



Bullen.



Guide for use with Acti brief Plus


Step 1.

Pull on the briefs, and pull the penis through the aperture whilst pushing the flange plate firmly against the groin.




Step 2.

Take 1 pouch and align both ring seals and push the two together creating a solid seal.

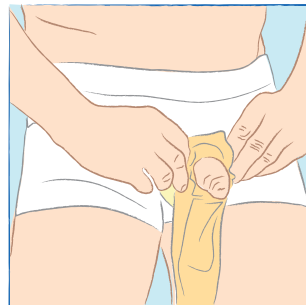

Step 3.

Expel any air that may have been trapped when applying the pouch and close the drain plug. To drain remove the drain plug and drain as normal. Close the drain plug when empty.

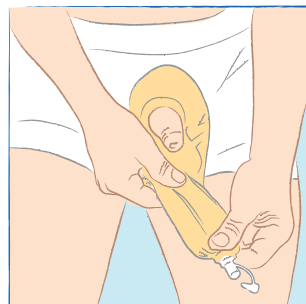

Leg / Night Bag attachment tube

Should you wish to increase the drainage capacity of the Acti brief Plus by use of a leg or night drainage bag, it is advisable to use this tube to obtain an effective seal between the Acti brief Plus outlet and your choice of bag.


Step 1.

Remove the stopper completely from the Acti brief Plus outlet tube.




Step 2.

Moisten and push the rubber tube over the outside of the Acti brief Plus outlet tube covering all the re-inforced section of the Acti brief Plus outlet tube.

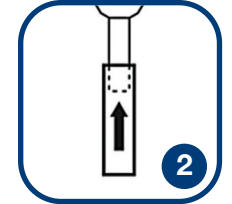

Step 3.

Push the other end of the connecting tube over the inlet tube of your chosen bag, sufficient for a good seal.

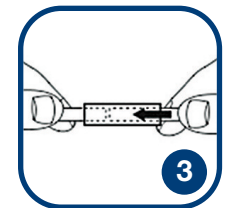

Step 4.

Gently try to pull the inlet and outlet tubes apart to test the seal you have obtained. Ideally the connecting tube should stretch before the seals on the outlet and inlet tubes move to release.

