

BULLEN

Size



▲ 20 mm ▲

▲ 25 mm ▲

▲ 27.5 mm ▲

▲ 30 mm ▲

▲ 32.5 mm ▲

▲ 35 mm ▲

▲ 40 mm ▲

Assert gentle pressure
to ascertain correct
flange size



Bullen.

PRESCRIPTION CODES

Spare pouches

SPlus18

SP/18

**Acti brief Plus
system**

ABPlus/18/20

ABPlus/18/25

ABPlus/18/27.5

ABPlus/18/30

ABPlus/18/32.5

ABPlus/18/35

ABPlus/18/40

acti brief **plus**⁺

Measuring guide for use with the Acti brief Plus systems

A comprehensive range of sizes ensure a snug & secure fit for complete peace of mind.

The Acti brief Plus system has been designed for use by any male urinary incontinence sufferer who does not use an indwelling catheter and therefore requires an external catheter solution or, as an alternative to absorbent pads.

Choosing the correct flange size will ensure a snug, but not tight fit around the base of the penis.

Too tight could cause restriction.

Too loose will allow backflow & leakage.

For further information on the Acti brief Plus system contact:

BULLEN HEALTHCARE on
FREEPHONE 0800 888 501

www.bullens.com

BULLEN

Size

▲ 20 mm ▲

▲ 25 mm ▲

▲ 27.5 mm ▲

▲ 30 mm ▲

▲ 32.5 mm ▲

▲ 35 mm ▲

▲ 40 mm ▲

Assert gentle pressure
to ascertain correct
flange size



acti brief **plus**⁺
TM